



**HIV Seroconversion Panel Donor No. 73695
Part Number: HIV12007**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV 1/2	GENE-PROBE PROCLEIX HIV-1/HCV ASSAY	ROCHE COBAS AMPLICOR HIV-1 MONITOR	VIROMED PCR	ABBOTT HIV-1 p24 Ag	ABBOTT IMx HIV 1/2 III	ABBOTT AxSYM HIV Ab 1/2 EIA	ABBOTT MUREX HIV Ab 1/2 + O (s/Co)
		TEST DATE: 8/6/2015	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 7/10/2000	TEST DATE: 6/30/2000	TEST DATE: 9/17/2002	TEST DATE: 9/10/2002	TEST DATE: 9/16/2002
12007-01	06/27/99	0.15	NEGATIVE	NEGATIVE	0	0.310	0.49	0.41	0.25
12007-02	06/29/99	0.14	NEGATIVE	NEGATIVE	0	0.293	0.43	0.41	0.22
12007-03	08/20/99	0.12	NEGATIVE	NEGATIVE	0	0.293	0.70	0.59	0.21
12007-04	10/22/99	0.3	POSITIVE	316000	75815	0.897	0.54	1.95	0.26
12007-05	10/24/99	2.97	POSITIVE	118000	63710	0.862	1.10	2.28	1.18
12007-06	10/29/99	60.1	POSITIVE	54600	14749	0.569	10.51	17.91	13.29
12007-07	10/31/99	55.3	POSITIVE	33500	29564	0.414	9.41	14.49	13.08
12007-08	11/05/99	38.7	POSITIVE	300000	18875	0.328	7.54	9.44	13.09
12007-09	11/07/99	32.8	POSITIVE	26400	23054	0.241	7.85	9.24	13.29

PANEL MEMBER	ORTHO VITROS ANTI- HIV 1/2	TRINITY BIOTECH UNIGOLD RECOMBIGNEN HIV-1	GENETICS SYSTEMS rLAV EIA	MEDMIRA ANTI-HIV-1 RAPID TEST	ORAQUICK ADVANCE HIV 1/2 Ab RAPID TEST	BIO-RAD HIV-1 WESTERN BLOT
	TEST DATE: 8/13/2002	TEST DATE: UNK	TEST DATE: 7/6/2000	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
12007-01	0.10	NEGATIVE	0.123	NEGATIVE	NEGATIVE	NOT TESTED
12007-02	0.09	NEGATIVE	0.130	NEGATIVE	NEGATIVE	NOT TESTED
12007-03	0.09	NEGATIVE	0.120	NEGATIVE	NEGATIVE	NEGATIVE
12007-04	0.17	NEGATIVE	0.113	NEGATIVE	NEGATIVE	NEGATIVE
12007-05	1.13	NEGATIVE	0.130	NEGATIVE	NEGATIVE	NEGATIVE
12007-06	30.30	POSITIVE	0.933	POSITIVE	NEGATIVE	P55,P40,P24,P18+/-
12007-07	27.40	POSITIVE	1.303	POSITIVE	NEGATIVE	GP160+/-,P55,P40,P24,P18+/-
12007-08	20.90	POSITIVE	1.655	POSITIVE	POSITIVE	GP160+/-,P65,P55,GP41+/-,P40,P31+/-,P24,P18
12007-09	21.20	POSITIVE	1.796	POSITIVE	POSITIVE	GP160+/-,P65,P55,GP41+/-,P40,P31+/-,P24,P18

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 73695 Part Number: HIV12007

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT PRISM HIV Ag/Ab COMBO TEST DATE: UNK	ABBOTT MUREX MICROPLATE HIV Ag/Ab COMBO TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1/2 RAPID TEST TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1 RAPID TEST TEST DATE: UNK	BIO-RAD MULTISPOT HIV 2 RAPID TEST TEST DATE: UNK
12007-01	0.07	0.276	0.09	NEGATIVE	NEGATIVE	NEGATIVE
12007-02	0.09	0.276	0.13	NEGATIVE	NEGATIVE	NEGATIVE
12007-03	0.06	0.256	0.12	NEGATIVE	NEGATIVE	NEGATIVE
12007-04	2.33	1.281	3.94	NEGATIVE	NEGATIVE	NEGATIVE
12007-05	2.10	1.882	3.35	NEGATIVE	NEGATIVE	NEGATIVE
12007-06	4.71	14.773	14.72	POSITIVE	NEGATIVE	NEGATIVE
12007-07	3.53	12.808	13.50	POSITIVE	NEGATIVE	NEGATIVE
12007-08	2.23	14.419	7.79	POSITIVE	POSITIVE	NEGATIVE
12007-09	2.11	12.813	5.63	POSITIVE	POSITIVE	NEGATIVE

PANEL MEMBER	ABBOTT HIV Ab ½ EIA rDNA (OD/Co) TEST DATE: UNK	ABBOTT PRISM ANTI-HIV ½ +O TEST DATE: 7/15/2002	BIOMERIEUX VIRONOSTIKA ANTI-HIV-1 TEST DATE: UNK	BIO-RAD ANTI-HIV-1 rLAV TEST DATE: UNK	BIOTEST ANTI-HIV TETRA ELISA TEST DATE: 7/1/2002	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 7/1/2002	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 6/24/2002
12007-01	0.095	0.28	0.485	0.069	0.12	0.03	0.00
12007-02	0.129	0.30	0.453	0.055	0.14	0.04	0.00
12007-03	0.121	0.27	0.515	0.044	0.15	0.05	0.00
12007-04	0.129	0.33	0.390	0.047	0.21	0.04	0.03
12007-05	1.147	0.60	0.347	0.051	0.89	0.39	1.23
12007-06	17.241	17.69	0.402	0.724	9.17	OVER	24.40
12007-07	17.241	17.87	0.669	1.204	9.32	OVER	24.14
12007-08	8.379	13.49	0.653	1.764	6.95	7.01	20.00
12007-09	6.957	18.07	0.656	2.309	6.30	7.18	18.10

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV12007 Rev06
Effective Date: 10/26/2021