



**HIV Seroconversion Panel Donor No. 65389  
Part Number: HIV9012**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA TEST DATE: 8/20/1998	ABBOTT HIV-1 p24 Ag TEST DATE: 8/26/1998	ABBOTT HIV-1 p24 Ag NT% TEST DATE: 9/3/1998	COULTER HIV-1 p24 Ag TEST DATE: 10/3/1998	ZEPTOMETRIX HIV p24 Ag (pg/mL) TEST DATE: UNK	ABBOTT AxSYM HIV AB 1/2 EIA TEST DATE: 9/16/1998	ABBOTT HIV AB 1/2 EIA rDNA TEST DATE: 7/21/1998	ABBOTT MUREX HIV Ab 1/2 + O 3 <sup>RD</sup> Generation TEST DATE: UNK
9012-01	11/12/97	157	0.51	NEGATIVE	0.08	<3.0	0.35	0.256	0.492
9012-02	11/14/97	<50	0.38	NEGATIVE	0.08	<3.0	0.33	0.256	0.456
9012-03	11/19/97	69	0.51	NEGATIVE	0.08	<3.0	0.31	0.171	0.452
9012-04	11/21/97	221	0.38	NEGATIVE	0.08	<3.0	0.33	0.171	0.403
9012-05	11/26/97	33740	0.63	NEGATIVE	1.05	<3.0	0.34	0.171	0.420
9012-06	11/28/97	101800	1.39	104.60	4.02	<3.0	0.31	0.256	0.570
9012-07	12/03/97	>500000	14.81	98.60	50.47	99.0	3.83	7.607	11.164
9012-08	12/05/97	>500000	>25.32	97.90	59.73	>125	7.20	>17.094	11.226

PANEL MEMBER	ORTHO VITROS Eci ANTI-HIV 1/2 TEST DATE: 3/18/2015	ABBOTT PRISM ANTI-HIV 1/2 TEST DATE: 8/18/1998	BIOMERIEUX ORGANON TEKNIKA ANTI-HIV-1 TEST DATE: UNK	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0 TEST DATE: 9/2/1998	BIOTEST ANTI-HIV 1/2 TEST DATE: 9/24/1998	DADE BEHRING ENZYGNOST ANTI-HIV 1/2 PLUS TEST DATE: 8/31/1998	GENETICS SYSTEMS HIV 1/2 EIA TEST DATE: 8/25/1998	GENETICS SYSTEMS HIV 1/2 +O TEST DATE: UNK	ORTHO ANTI-HIV 1/2 ENHANCED ELISA TEST DATE: 9/15/1998
9012-01	0.19	0.22	0.52	0.42	0.052	0.03	0.192	0.178	0.02
9012-02	0.19	0.25	0.50	0.39	0.060	0.04	0.224	0.236	0.03
9012-03	0.17	0.24	0.50	0.35	0.057	0.03	0.169	0.168	0.01
9012-04	0.17	0.27	0.50	0.38	0.052	0.02	0.160	0.181	0.08
9012-05	0.20	0.26	0.57	0.46	0.015	0.02	0.163	0.223	0.05
9012-06	1.53	0.24	0.57	0.36	0.054	0.03	0.141	0.223	0.90
9012-07	62.9	1.09	0.68	2.90	1.696	3.90	0.188	9.709	MAX ABS.
9012-08	69.8	3.13	0.97	3.44	2.887	MAX ABS.	0.642	9.709	MAX ABS.

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9012 Rev06  
Effective Date: 03/24/2022



**HIV Seroconversion Panel Donor No. 65389  
Part Number: HIV9012**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	TRINITY BIOTECH UNIGOLD RECOMBI GEN HIV-1  TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1 RAPID TEST  TEST DATE: UNK	BIO-RAD MULTISPOT HIV 2 RAPID TEST  TEST DATE: UNK	ORAQUICK ADVANCE HIV ½ TEST  TEST DATE: UNK	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT  TEST DATE: UNK	ABBOTT PRISM HIV Ag/Ab COMBO 4 <sup>th</sup> Generation  TEST DATE: UNK	ABBOTT MUREX HIV Ag/Ab COMBO 4 <sup>th</sup> Generation  TEST DATE: UNK	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 <sup>th</sup> Generation  TEST DATE: UNK
9012-01	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	0.11	0.240	0.10
9012-02	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	0.09	0.255	0.07
9012-03	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	0.07	0.260	0.08
9012-04	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	0.08	0.245	0.11
9012-05	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	0.32	0.540	0.37
9012-06	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	0.73	1.415	1.02
9012-07	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	17.44	17.655	15.17
9012-08	POSITIVE	POSITIVE	NEGATIVE	NEGATIVE	P24,P160 (VF)	46.55	17.675	49.99

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)