

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 65685 Part Number: HIV9019

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA TEST DATE: 8/20/1998	ABBOTT HIV-1 p24 Ag TEST DATE: 7/21/1998	ABBOTT HIV-1 p24 Ag NT% TEST DATE: 9/3/1998	COULTER HIV-1 p24 Ag TEST DATE: 8/26/1998	ABBOTT AxsYM HIV AB ½ EIA TEST DATE: 9/16/1998	ABBOTT HIV AB ½ EIA rDNA TEST DATE: 7/21/1998	ABBOTT MUREX HIV Ab ½ + O 3 RD Generation TEST DATE: UNK
9019-01	11/20/97	<50	0.317	NEGATIVE	0.64	0.35	0.256	0.314
9019-02	11/23/97	<50	0.317	NEGATIVE	0.56	0.39	0.171	0.258
9019-03	12/28/97	571700	1.571	NEGATIVE	1.44	7.54	>17.094	12.728

PANEL MEMBER	ORTHO VITROS ECi ANTI-HIV 1/2 TEST DATE: 3/23/2015	ABBOTT PRISM ANTI-HIV ½ TEST DATE: 8/18/1998	BIOMERIEUX ORGANON TEKNIKA ANTI-HIV-1 TEST DATE: UNK	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0 TEST DATE: 9/2/1998	BIOTEST ANTI-HIV ½ TEST DATE: 9/24/1998	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 8/31/1998	GENETICS SYSTEMS HIV ½ EIA TEST DATE: 8/25/1998
9019-01	0.15	0.23	0.50	0.43	0.052	0.02	0.150
9019-02	0.16	0.26	0.50	0.83	0.018	0.01	0.150
9019-03	21.0	70.38	2.33	7.67	6.797	5.37	8.147

PANEL MEMBER	GENETICS SYSTEMS HIV ½ + O TEST DATE: UNK	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 9/15/1998	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT TEST DATE: UNK	ORAQUICK ADVANCE HIV ½ RAPID Ab TEST DATE: UNK	TRINITY BIOTECH UNIGOLD RECOMBIGN HIV-1 TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1 RAPID TEST TEST DATE: UNK	BIO-RAD MULTISPOT HIV 2 RAPID TEST TEST DATE: UNK	ABBOTT PRISM HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK
9019-01	0.168	0.05	P24 (VF)	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	0.17
9019-02	0.136	0.05	P24 (VF)	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	0.06
9019-03	9.709	11.0	P24 P55 P66 P120	POSITIVE	POSITIVE	POSITIVE	NEGATIVE	10.54

PANEL MEMBER	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 TH Generation TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	BIORAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK
9019-01	0.259	0.08	0.16	0.16	0.05
9019-02	0.234	0.07	0.07	0.5	0.04
9019-03	17.934	57.28	129.83	NOT REACTIVE-HIGH AB	129.83

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9019 Rev05
Effective Date: 10/25/2021