

# ZeptoMetrix®

## HIV Seroconversion Panel Donor No. 64578 Part Number: HIV9022

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA TEST DATE: 10/27/1999	ABBOTT HIV-1 p24 Ag TEST DATE: 6/16/1999	COULTER HIV-1 p24 Ag TEST DATE: 6/10/1999	ROCHE ELECSYS HIV p24 Ag TEST DATE: 6/21/1999	ABBOTT AxSYM HIV AB ½ EIA TEST DATE: 7/14/1999	ABBOTT HIV AB ½ EIA rDNA TEST DATE: 5/11/1999	ABBOTT HIV AB ½ EIA rDNA 3 <sup>rd</sup> Generation TEST DATE: 7/12/1999	ABBOTT MUREX HIV Ab ½ + O 3 <sup>rd</sup> Generation TEST DATE: UNK
9022-01	07/30/97	<50	0.41	0.241	0.37	0.24	0.121	0.34	0.384
9022-02	08/02/97	<50	0.39	0.266	0.32	0.21	0.121	0.33	0.350
9022-03	08/06/97	<50	0.40	0.253	0.31	0.23	0.129	0.28	0.341
9022-04	08/09/97	<50	0.41	0.342	0.32	0.26	0.138	0.26	0.306
9022-05	08/14/97	<50	0.48	0.253	0.51	0.21	0.138	0.26	0.341
9022-06	08/16/97	107	0.43	0.177	0.58	0.19	0.129	0.34	0.394
9022-07	08/22/97	32296	0.51	0.544	1.33	0.23	0.103	0.27	0.647
9022-08	08/24/97	199644	1.94	3.051	7.45	0.21	0.129	0.26	0.431
9022-09	08/31/97	>500000	>24.4	36.329	320.9	10.80	14.828	>14.00	10.631

PANEL MEMBER	ABBOTT HIV AB ½ EIA rDNA IMx TEST DATE: 7/19/1999	ABBOTT MUREX SUDS ANTI-HIV-1 TEST DATE: 7/21/1999	ABBOTT PRISM ANTI-HIV ½ TEST DATE: 7/15/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV-1 MICROELISA SYSTEM TEST DATE: 6/16/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0 TEST DATE: 6/9/1999	BIO-RAD NOVAPATH HIV-1 IMMUNOBLOT TEST DATE: UNK	BIOTEST ANTI-HIV ½ TEST DATE: 6/9/1999	BIOTEST ANTI-HIV-1 TETRA ELISA TEST DATE: 6/8/1999
9022-01	0.36	0.15	0.26	0.202	0.48	NEGATIVE	0.04	0.29
9022-02	0.33	0.17	0.27	0.231	0.42	NEGATIVE	0.06	0.28
9022-03	0.39	0.16	0.26	0.231	0.45	NEGATIVE	0.03	0.26
9022-04	0.34	0.17	0.28	0.231	0.45	NEGATIVE	0.03	0.34
9022-05	0.31	0.19	0.28	0.259	0.46	NEGATIVE	0.05	0.26
9022-06	0.36	0.17	0.30	0.231	0.52	NEGATIVE	0.04	0.27
9022-07	0.28	0.15	0.25	0.231	0.42	NEGATIVE	0.03	0.25
9022-08	0.33	0.16	0.31	0.259	0.48	NEGATIVE	0.04	0.25
9022-09	11.71	9.30	8.79	0.288	3.00	NEGATIVE	0.71	MAX ABS.

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9022 Rev08  
Effective Date: 06/08/2021

# ZeptoMetrix®

## HIV Seroconversion Panel Donor No. 64578 Part Number: HIV9022

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 6/8/1999	GENETICS SYSTEMS ANTI-HIV ½ PEPTIDE EIA TEST DATE: 6/10/1999	GENETICS SYSTEMS rLAV EIA TEST DATE: 11/15/1999	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 6/29/1999	ROCHE ENZYMUM ANTI-HIV-COMBI TEST DATE: 7/14/1999	SANOFI GENSCREEN ANTI-HIV ½ TEST DATE: 6/21/1999	ABBOTT PRISIM HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 <sup>TH</sup> Generation TEST DATE: UNK
9022-01	0.14	0.246	0.071	0.05	0.10	0.04	0.07	0.249
9022-02	0.13	0.429	0.074	0.04	0.09	0.09	0.08	0.213
9022-03	0.14	0.314	0.083	0.06	0.09	0.09	0.09	0.213
9022-04	0.13	0.429	0.080	0.06	0.09	0.09	0.08	0.228
9022-05	0.12	0.257	0.080	0.05	0.08	0.04	0.06	0.264
9022-06	0.15	0.314	0.062	0.06	0.09	0.08	0.06	0.254
9022-07	0.14	0.257	0.053	0.05	0.10	0.08	0.51	0.690
9022-08	0.14	0.200	0.062	0.06	0.56	0.06	5.09	3.792
9022-09	6.11	0.229	0.133	12.93	11.80	MAX-ABS	126.98	18.954

PANEL MEMBER	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation TEST DATE: UNK	ORTHO VITROS ECi HBsAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HCV TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HIV ½ TEST DATE: 8/30/2018
9022-01	NOT TESTED	0.14	0.14	0.10	0.05	0.03	0.10
9022-02	0.10	0.07	0.07	0.03	0.05	0.03	0.10
9022-03	0.14	0.09	0.09	0.00	0.05	0.02	0.10
9022-04	0.08	0.04	0.04	0.03	0.05	0.03	0.10
9022-05	0.08	0.07	0.07	0.01	0.07	0.03	0.10
9022-06	0.15	0.07	0.05	0.07	0.06	0.02	0.10
9022-07	NOT TESTED	2.42	2.42	0.04	0.06	0.03	0.10
9022-08	4.62	26.90	26.90	0.03	0.06	0.02	0.11
9022-09	175.31	>200	>200	1.45	0.06	0.02	37.0

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9022 Rev08  
Effective Date: 06/08/2021