



**HIV Seroconversion Panel Donor No. 68582  
Part Number: HIV9030**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON	ABBOTT	COULTER	ROCHE	ZEPTOMETRIX	ABBOTT	ABBOTT
		HIV-1 bDNA	HIV-1 p24 Ag	HIV-1 p24 Ag	ELECSYS HIV p24Ag	p24 Ag (pg/mL)	AxSYM HIV AB ½ EIA	HIV AB ½ EIA rDNA
		TEST DATE: 10/26/1999	TEST DATE: 5/4/1999	TEST DATE: 8/3/1999	TEST DATE: 7/6/1999	TEST DATE: UNK	TEST DATE: 7/14/1999	TEST DATE: 5/4/1999
9030-01	08/14/98	<50	0.45	0.157	0.60	<3.0	0.22	0.099
9030-02	08/18/98	<50	0.45	0.165	0.59	<3.0	0.24	0.081
9030-03	08/21/98	<50	0.44	0.174	0.60	<3.0	0.30	0.090
9030-04	08/25/98	<50	0.46	0.130	0.53	<3.0	0.25	0.081
9030-05	08/28/98	<50	0.49	0.122	0.63	<3.0	0.26	0.081
9030-06	09/01/98	<50	0.57	0.148	0.63	<3.0	0.27	0.108
9030-07	09/04/98	<50	0.51	0.139	0.56	<3.0	0.25	0.126
9030-08	09/08/98	<50	0.48	0.174	0.58	<3.0	0.27	0.090
9030-09	09/11/98	<50	0.44	0.130	0.62	<3.0	0.25	0.081
9030-10	09/16/98	<50	0.43	0.191	0.58	<3.0	0.22	0.090
9030-11	09/18/98	<50	0.41	0.209	0.61	<3.0	0.22	0.099
9030-12	09/23/98	85	0.46	0.130	0.66	<3.0	0.25	0.117
9030-13	09/25/98	1317	0.45	0.139	0.72	<3.0	0.26	0.099
9030-14	09/30/98	238947	1.49	10.348	10.23	6.6	0.25	0.108
9030-15	10/02/98	>500000	2.98	19.043	20.75	15.2	0.33	0.396
9030-16	10/07/98	>500000	4.71	29.217	49.54	56.6	10.63	>18.018

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9030 Rev05  
Effective Date: 10/26/2021



**HIV Seroconversion Panel Donor No. 68582  
Part Number: HIV9030**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT HIV AB ½ EIA rDNA 3 <sup>rd</sup> Generation TEST DATE: 7/12/1999	ABBOTT MUREX HIV Ab ½ + O 3 <sup>rd</sup> Generation TEST DATE: UNK	ABBOTT HIV AB ½ EIA rDNA IMx TEST DATE: 7/19/1999	ABBOTT MUREX HIV ½ ICE+ O TEST DATE: 7/21/1999	ABBOTT PRISM ANTI-HIV ½ TEST DATE: 7/15/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV-1 MICROELISA SYSTEM TEST DATE: UNK	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0 TEST DATE: 6/9/1999	BIO-RAD NOVAPATH HIV-1 IMMUNOBLOT TEST DATE: UNK
9030-01	0.28	0.344	0.31	0.19	0.33	0.470	0.39	NEGATIVE
9030-02	0.26	0.295	0.32	0.19	0.27	0.491	0.44	NEGATIVE
9030-03	0.32	0.257	0.33	0.18	0.21	0.449	0.42	NEGATIVE
9030-04	0.28	0.243	0.35	0.19	0.26	0.449	0.48	NEGATIVE
9030-05	0.28	0.253	0.37	0.18	0.77	0.449	0.41	NEGATIVE
9030-06	0.30	0.257	0.28	0.15	0.37	0.470	0.43	NEGATIVE
9030-07	0.29	0.243	0.41	0.17	0.27	0.513	0.60	NEGATIVE
9030-08	0.27	0.274	0.37	0.18	0.24	0.470	0.46	NEGATIVE
9030-09	0.31	0.309	0.38	0.22	0.22	0.427	0.43	NEGATIVE
9030-10	0.24	0.288	0.36	0.17	0.22	0.470	0.45	NEGATIVE
9030-11	0.35	0.240	0.36	0.18	0.23	0.470	0.40	NEGATIVE
9030-12	0.50	0.243	0.41	0.19	0.25	0.449	0.45	NEGATIVE
9030-13	0.26	0.247	0.37	0.17	0.22	0.449	0.43	NEGATIVE
9030-14	0.26	0.278	0.38	0.17	0.29	0.449	0.44	NEGATIVE
9030-15	0.62	0.503	0.51	0.24	0.34	0.449	0.50	NEGATIVE
9030-16	>14.0	10.326	16.09	3.65	30.27	0.620	1.80	NEGATIVE

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9030 Rev05  
Effective Date: 10/26/2021



**HIV Seroconversion Panel Donor No. 68582  
Part Number: HIV9030**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BIOTEST ANTI-HIV ½  TEST DATE: 6/9/1999	BIOTEST ANTI-HIV-1 TETRA ELISA  TEST DATE: 6/8/1999	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS  TEST DATE: 6/8/1999	GENETICS SYSTEMS HIV ½ PEPTIDE EIA  TEST DATE: 7/29/1999	GENETICS SYSTEMS rLAV EIA  TEST DATE: 11/15/1999	ORTHO ANTI-HIV ½ ENHANCE D ELISA  TEST DATE: 6/29/1999	ROCHE ENZYMMUM ANTI-HIV-1 COMBI  TEST DATE: 7/14/1999	SANOFI GENSCORE EN ANTI-HIV ½  TEST DATE: 6/21/1999	ABBOTT PRISIM HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation  TEST DATE: UNK
9030-01	0.06	0.24	0.16	0.188	0.147	0.06	0.09	0.05	0.09
9030-02	0.02	0.23	0.17	0.156	0.147	0.05	0.10	0.05	0.06
9030-03	0.03	0.25	0.14	0.156	0.118	0.05	0.09	0.05	0.10
9030-04	0.03	0.26	0.15	0.156	0.130	0.06	0.09	0.05	0.07
9030-05	0.05	0.24	0.15	0.188	0.118	0.06	0.09	0.08	0.07
9030-06	0.02	0.23	0.14	0.156	0.109	0.06	0.08	0.14	0.08
9030-07	0.04	0.25	0.14	0.125	0.097	0.04	0.08	0.11	0.08
9030-08	0.03	0.23	0.14	0.156	0.077	0.05	0.08	0.10	0.06
9030-09	0.04	0.26	0.18	0.188	0.112	0.05	0.09	0.11	0.05
9030-10	0.05	0.24	0.15	0.219	0.132	0.06	0.08	0.06	0.05
9030-11	0.04	0.25	0.12	0.156	0.107	0.06	0.08	0.06	0.06
9030-12	0.02	0.24	0.14	0.281	0.091	0.06	0.09	0.06	0.10
9030-13	0.04	0.26	0.14	0.472	0.114	0.05	0.08	0.08	0.08
9030-14	0.04	0.31	0.15	0.188	0.126	0.17	0.58	0.09	3.73
9030-15	0.05	1.36	0.22	0.156	0.107	1.824	0.88	0.56	10.72
9030-16	1.00	8.34	5.90	0.406	0.502	MAX ABS	10.66	MAX ABS	24.82

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9030 Rev05  
Effective Date: 10/26/2021



**HIV Seroconversion Panel Donor No. 68582  
Part Number: HIV9030**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 <sup>TH</sup> Generation TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation TEST DATE: UNK	ORTHO VITROS ECi ANTI-HIV ½ TEST DATE: 8/30/2018
9030-01	0.253	0.07	0.08	0.08	0.01	0.11
9030-02	0.242	0.10	0.05	0.05	0.02	0.11
9030-03	0.232	0.35	0.08	0.05	0.05	0.11
9030-04	0.237	0.25	0.02	0.02	0.01	0.12
9030-05	0.232	0.17	0.07	0.07	0.02	0.12
9030-06	0.242	0.18	0.03	0.03	0.02	0.12
9030-07	0.242	0.14	0.06	0.06	0.04	0.12
9030-08	0.242	0.27	0.12	0.12	0.01	0.12
9030-09	0.242	0.16	0.08	0.08	0.04	0.11
9030-10	0.227	0.13	0.09	0.09	0.04	0.12
9030-11	0.232	0.34	0.05	0.03	0.04	0.13
9030-12	0.227	0.16	0.06	0.06	0.05	0.12
9030-13	0.258	0.09	0.12	0.12	0.01	0.11
9030-14	4.702	3.84	13.34	13.34	0.01	0.20
9030-15	11.434	15.10	37.26	37.26	0.05	1.87
9030-16	17.869	41.91	70.94	70.94	2.91	26.0

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9030 Rev05  
Effective Date: 10/26/2021