



HIV Seroconversion Panel Donor No. 66632
Part Number: HIV9034

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA	ABBOTT HIV-1 p24 Ag	ROCHE ENZYMUM ANTI-HIV-1 COMBI	COULTER HIV-1 p24 Ag	ABBOTT AxSYM HIV AB ½ EIA	ABBOTT HIV AB ½ EIA rDNA	ABBOTT HIV Ab ½ EIA rDNA 3 RD Generation	ABBOTT HIV Ab ½ EIA rDNA IMx
		TEST DATE: 10/27/1999	TEST DATE: 8/29/2000	TEST DATE: 11/15/1999	TEST DATE: 8/3/1999	TEST DATE: 11/17/1999	TEST DATE: 5/11/1999	TEST DATE: 11/15/1999	TEST DATE: 6/7/2000
9034-01	02/01/98	<50	0.31	0.24	0.148	0.31	0.129	0.20	0.40
9034-02	02/03/98	<50	0.33	0.21	0.104	0.22	0.095	0.23	0.38
9034-03	02/08/98	<50	0.30	0.22	0.139	0.22	0.121	0.21	0.41
9034-04	02/10/98	<50	0.28	0.26	0.287	0.28	0.129	0.21	0.38
9034-05	02/24/98	<50	0.33	0.28	0.122	0.21	0.138	0.20	0.35
9034-06	02/26/98	<50	0.30	0.29	0.139	0.21	0.112	0.20	0.38
9034-07	03/05/98	<50	0.33	0.30	0.130	0.24	0.155	0.21	0.37
9034-08	03/07/98	<50	0.31	0.28	0.130	0.23	0.129	0.21	0.38
9034-09	03/12/98	<50	0.30	0.31	0.122	0.25	0.129	0.23	0.36
9034-10	03/14/98	54	0.33	0.29	0.130	0.22	0.147	0.25	0.43
9034-11	03/19/98	123310	0.87	0.34	2.191	0.69	0.147	0.23	0.47
9034-12	03/23/98	>500000	6.00	1.07	24.609	0.38	0.379	0.32	0.65
9034-13	03/26/98	>500000	11.08	2.44	29.391	0.88	0.974	1.07	1.43

PANEL MEMBER	ABBOTT MUREX HIV Ab ½ + O 3 RD Generation	ABBOTT MUREX SUDS ANTI HIV-1	ABBOTT PRISM ANTI-HIV ½	BIOMERIEUX ORGANON VIRNOSTIKA HIV-1 MICRO ELISA SYSTEM	BIOR-RAD NOVAPATH HIV-1 IMMUNOBLOT	DADE BEHRING ENZYGNOST INTEGRAL HIV	GENETICS SYSTEMS HIV ½ PEPTIDE EIA	ORTHO ANTI-HIV ½ ENHANCED ELISA	ORTHO VITROS ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: 8/21/2000	TEST DATE: 9/11/2000	TEST DATE: 8/3/1999	TEST DATE: 9/4/2000	TEST DATE: 11/17/1999	TEST DATE: 8/4/1999	TEST DATE: 4/4/2000	TEST DATE: 8/30/2018
9034-01	0.351	0.32	0.25	0.496	NEGATIVE	0.02	0.100	0.05	0.13
9034-02	0.306	0.27	0.23	0.504	NEGATIVE	0.03	0.118	0.05	0.13
9034-03	0.330	0.29	0.25	0.515	NEGATIVE	0.03	0.118	0.05	0.12
9034-04	0.281	0.27	0.27	0.515	NEGATIVE	0.02	0.118	0.05	0.12
9034-05	0.858	0.26	0.27	0.433	NEGATIVE	0.05	0.088	0.06	0.15
9034-06	0.285	0.30	0.25	0.470	NEGATIVE	0.02	0.206	0.07	0.14
9034-07	0.330	0.35	0.37	0.519	NEGATIVE	0.02	0.206	0.07	0.14
9034-08	0.389	0.38	0.33	0.444	NEGATIVE	0.02	0.088	0.05	0.13
9034-09	0.389	0.33	0.31	0.481	NEGATIVE	0.02	0.176	0.05	0.13
9034-10	0.337	0.29	0.32	0.456	NEGATIVE	0.03	0.176	0.05	0.13
9034-11	0.365	0.34	0.27	0.430	NEGATIVE	0.08	0.118	0.04	0.13
9034-12	0.420	0.39	0.41	0.537	NEGATIVE	0.75	0.206	0.19	0.18
9034-13	1.090	0.77	1.34	1.000	NEGATIVE	1.28	0.176	1.45	1.22

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9034 Rev06
 Effective Date: 06/14/2021



**HIV Seroconversion Panel Donor No. 66632
Part Number: HIV9034**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT PRISM HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 TH Generation TEST DATE: UNK	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 TH Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK
9034-01	0.07	0.237	0.15	0.15	0.13	0.11
9034-02	0.12	0.227	0.13	0.08	0.08	0.05
9034-03	0.07	0.566	0.11	0.10	0.10	0.02
9034-04	0.07	0.217	0.10	0.14	0.14	0.04
9034-05	0.10	0.217	0.07	0.09	0.09	0.04
9034-06	0.07	0.222	0.12	0.07	0.07	0.01
9034-07	0.10	0.273	0.10	0.09	0.09	0.03
9034-08	0.10	0.258	0.15	0.13	0.13	0.06
9034-09	0.07	0.263	0.14	0.09	0.09	0.06
9034-10	0.07	0.253	0.10	0.12	0.12	0.08
9034-11	1.09	1.394	2.28	6.14	6.14	0.03
9034-12	17.64	14.601	29.75	84.03	84.03	0.05
9034-13	22.86	17.636	41.00	144.01	144.01	0.16

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9034 Rev06
Effective Date: 06/14/2021