

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 63753 Part Number: HIV9076

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	GEN- PROBE PROCLEIX HIV-1 / HCV ASSAY	ROCHE COBAS AMPLICOR HIV-1 MONITOR	ABBOTT HIV-1 p24 Ag	COULTER HIV-1 p24 Ag	ZEPTOMETRIX HIV p24 Ag (pg/mL)
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9076-01	02/21/97	NEGATIVE	NOT DETECTED	NOT TESTED	0.247	<3.0
9076-02	03/05/97	NEGATIVE	NOT DETECTED	NOT TESTED	0.247	<3.0
9076-03	03/22/97	NEGATIVE	NOT DETECTED	NOT TESTED	0.247	<3.0
9076-04	03/29/97	NEGATIVE	NOT DETECTED	NOT TESTED	0.123	<3.0
9076-05	04/14/97	POSITIVE	NOT DETECTED	NOT TESTED	0.37	<3.0
9076-06	04/20/97	POSITIVE	13,400	NOT TESTED	0.617	<3.0
9076-07	04/22/97	NOT TESTED	1500	0.24	1.327	NOT TESTED
9076-08	04/28/97	POSITIVE	>750000	3.97	13.58	98.8
9076-09	05/01/97	POSITIVE	>750000	15.87	16.04	78.8
9076-10	05/06/97	POSITIVE	18500	15.87	31.6	4.0

PANEL MEMBER	ABBOTT HIV Ab ½ EIA rDNA 3 rd Generation	ABBOTT MUREX HIV Ab ½ + O 3 rd Generation	ORTHO VITROS ECI ANTI-HIV 1/2	BIOMERIEUX VIRNOSTIKA ANTI-HIV-1	BIO-RAD ANTI-HIV-1 rLAV	GENETICS SYSTEMS HIV ½ + O
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/4/2015	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9076-01	0.155	0.539	0.14	0.558	0.052	0.154
9076-02	0.147	0.548	0.14	0.767	0.122	0.177
9076-03	0.164	0.427	0.13	0.539	0.100	0.188
9076-04	0.103	0.310	0.14	0.384	0.089	0.249
9076-05	0.164	0.313	0.14	0.408	0.092	0.150
9076-06	0.155	0.365	0.12	0.472	0.100	0.556
9076-07	NOT TESTED	0.412	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9076-08	4.879	2.876	16.3	0.324	0.133	1.348
9076-09	17.241	10.477	39.4	0.579	0.764	5.543
9076-10	17.241	6.031	30.3	1.812	4.000	10.239

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9076 Rev06
Effective Date: 10/26/2021

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 63753 Part Number: HIV9076

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	MEDMIRA ANTI-HIV 1 RAPID TEST TEST DATE: UNK	TRINITY BIO-TECH UNI GOLD RECOMBIGNEN TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1 RAPID TEST TEST DATE: UNK	BIO-RAD MULTISPOT HIV 2 RAPID TEST TEST DATE: UNK	BIO-RAD HIV-1 WESTERN BLOT TEST DATE: UNK	ORAQUICK ADVANCE HIV ½ RAPID Ab TEST DATE: UNK
9076-01	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NOT TESTED	NEGATIVE
9076-02	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NOT TESTED	NEGATIVE
9076-03	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NOT TESTED	NEGATIVE
9076-04	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NOT TESTED	NEGATIVE
9076-05	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9076-06	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9076-07	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9076-08	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9076-09	POSITIVE	POSITIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9076-10	POSITIVE	POSITIVE	POSITIVE	NEGATIVE	gp160+/-,p55+/-,p40+/-,p24	POSITIVE

PANEL MEMBER	ABBOTT PRISM HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 th Generation TEST DATE: UNK	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK
9076-01	0.15	0.348	0.16	0.09	0.08	0.04
9076-02	0.12	0.309	0.11	0.09	0.08	0.07
9076-03	0.19	0.265	0.16	0.03	0.03	0.01
9076-04	0.08	0.304	0.09	0.10	0.10	0.05
9076-05	0.07	0.319	0.13	0.07	0.07	0.03
9076-06	0.17	0.358	0.28	0.11	0.11	0.04
9076-07	0.15	0.299	0.14	NOT TESTED	NOT TESTED	NOT TESTED
9076-08	22.47	15.990	31.29	95.89	95.89	0.11
9076-09	17.01	16.848	43.01	68.67	68.67	0.82
9076-10	1.39	9.917	3.59	49.99	43.07	49.99

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9076 Rev06
Effective Date: 10/26/2021