



## Hepatitis E Mixed Titer Panel

Product Code: KZMC003

Lot# 0802-272-00032

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures.*

**Application Notes:** The syphilis positive donor plasmas used for this panel were obtained under informed consent and collected in US FDA licensed plasma centers. Donors were untreated and asymptomatic at the time of collection. Positive test results below are highlighted in red.

PANEL MEMBER	LOT NUMBER	**ABBOTT AUZYME (HBsAg) (S/Co)	**ABBOTT AUSAB (Anti-HBs) (S/Co)	*ABBOTT CORZYME (HBc TOTAL) (S/Co)	**ABBOTT COR-M rDNA (HBc IgM) (S/Co)	**ABBOTT HIV 1/2 (S/Co)	**ABBOTT HCV 2.0 (S/Co)	FISHER SUREVUE RPR	**GENELABS HEV IgG (S/Co)	**GENELABS HEV IgM (S/Co)
1	9612-040-5652	0.10	1.8	2.6	0.05	0.2	0.3	NON-REACTIVE	4.8	0.04
2	9612-040-5654	0.20	2.2	2.6	0.05	0.2	0.3	NON-REACTIVE	4.1	0.40
3	9612-040-5658	0.04	4.2	2.8	0.05	0.2	0.2	NON-REACTIVE	2.7	0.09
4	9612-040-5660	0.04	0.1	2.2	0.10	0.3	0.2	NON-REACTIVE	2.0	0.20
5	9612-040-5661	0.04	0.5	2.2	0.05	0.2	0.2	NON-REACTIVE	2.6	0.20
6	9612-040-5665	0.02	0.1	2.2	0.05	0.2	0.5	NON-REACTIVE	1.4	0.40
5	9612-040-5671	0.10	0.1	2.5	0.05	0.2	0.3	NON-REACTIVE	5.5	0.02
8	9612-040-5672	0.10	0.1	2.5	0.05	0.2	0.3	NON-REACTIVE	5.7	0.40
9	9612-040-5675	0.05	0.3	2.4	0.05	0.4	0.2	NON-REACTIVE	3.1	0.04
10	9612-040-5677	0.10	0.4	1.9	0.09	0.2	0.2	NON-REACTIVE	1.3	0.09
11	9612-040-5678	0.04	0.4	2.1	0.05	0.2	0.3	NON-REACTIVE	2.1	0.09
12	9612-040-5679	0.02	0.3	2.1	0.05	0.4	0.3	NON-REACTIVE	1.9	0.30
13	9612-040-5680	0.02	1.7	2.4	0.05	0.2	0.4	NON-REACTIVE	2.0	0.50
14	9612-040-5681	0.02	4.7	2.5	0.05	0.2	0.2	NON-REACTIVE	3.1	0.04
15	9612-040-5682	0.04	0.1	1.9	0.05	0.2	0.3	NON-REACTIVE	1.7	0.20
16	9612-040-5683	0.02	0.1	2.1	0.30	0.2	0.3	NON-REACTIVE	2.1	0.02
17	9612-040-5684	0.04	0.1	2.4	0.05	0.2	0.2	NON-REACTIVE	2.9	0.02
18	9612-040-5685	0.02	0.1	2.1	0.05	0.2	0.2	NON-REACTIVE	3.9	0.20
19	9612-040-5686	0.04	0.1	2.4	0.09	0.2	0.3	NON-REACTIVE	5.1	0.10
20	9612-040-5687	0.05	0.1	2.1	0.05	0.2	0.3	NON-REACTIVE	2.6	0.30
21	9612-040-5688	0.07	0.3	2.2	0.05	0.2	0.3	NON-REACTIVE	3.7	0.20
22	9612-040-5690	0.07	0.1	2.5	0.05	0.2	0.2	NON-REACTIVE	3.5	0.09
23	9612-040-5691	0.04	0.1	2.5	0.05	0.2	0.3	NON-REACTIVE	5.3	0.02
24	9612-040-5692	0.04	0.7	2.6	0.05	0.2	0.2	NON-REACTIVE	1.3	0.09

\*S/Co >1.0 is considered negative S/Co < 1.0 is considered positive  
\*\* S/Co < 1.0 is considered negative S/Co > 1.0 is considered positive

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)